siare[®]

ENGINEERING
INTERNATIONAL GROUP s.r.l.

SIARETRON 1000 IPER

Lung ventilator
for hyperbaric chamber
*Ventilatore polmonare
per camera iperbarica*

NEPTUNE HBOT
Vital signs monitor
for hyperbaric chamber
*Monitor parametri vitali
per camera iperbarica*

SIARETRON 1000 IPER

20 years of proven experience
up to a depth of 60mH₂O

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The only advanced Intensive care ventilator for hyperbaric chambers designed for medical, diving and tunneling applications.

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High Performance Intensive care ventilator for HBOT in multiplace and monoplace chambers suitable for adults, children and newborns.

SIARETRON 1000 IPER the new
electronic lung ventilator
Now achieving its 5th generation has
been expressly conceived for any
hyperbaric application.

Main feature:

- user-friendly
- respiratory rate stability at any depth
- volume automatic compensation
- Oxygen concentration adjustable from 21 up to 100%
- **Dedicated program for newborns ventilation from 2 ml and up to 3000 ml for adults.**
- **Compatible with premixed HELIOX**

- **Extensive monitoring of any respiratory parameter including compensated volume and O2 concentration**
- Available **with remote control panel** for monoplace chamber installation and VGA connection for slave monitors outside the chamber
- Small, portable and light (5,5kg)
- **User friendly.**

Both the menu design and the parameters self-adjustment of this ventilator make its use simple and intuitive.

- **Respiratory rate compensation at any depth**

The use of a low power special circuit controlling the functions of the Ventilator grants the accuracy of the set parameters.

This stability cannot be compared with pneumatic ventilators.

- **Automatic volume compensation**

The rate stability is not enough to grant the perfect operation of the

ventilator since the variation of the gas density due to the compression, tends to decrease the tidal volume at the depth increasing. The electropneumatic module automatically modifies the flow according to the diving depth, which is measured by a special absolute pressure transducer.

This way, the ventilator delivers a constant volume at any depth (see graphic and scheme)

- **Max flow up to 240 l/min.**

Oxygen concentration adjustable from 21 up to 100%

Automatic safety system able to reduce the oxygen concentration at depth increasing.

Electric feeding .

Through low voltage power and **special hermetic battery.**

Built-in power supply unit

Gas Feeding.

Air and Oxygen at 3 bar above the operating chamber pressure

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NEPTUNE HBOT

Hyperbaric Vital Signs Monitor

- Matchless performance up to a depth of 60 mH₂O
- Monitored parameters:
ECG 3/5 lead - RESP - NIBP - TEMP (2 channels) - SPO₂ - ARR ANALYSIS
ST ANALYSIS
- NIBP technology based on a pneumatic valve supplied with compressed AIR
- Suitable for adults, children and newborns
- 12" TFT LED COLOUR MONITOR with HIGH RESOLUTION
- Small dimensions and light weight (3,5 kg)
- Special hermetic battery packs with autonomies from 2 to 10 hours

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SIARETRON 1000 IPER

- Matchless performance up to a depth of 60 mH₂O
- Automatic compensation of all ventilation parameters
- High performance Intensive care ventilator
- Adult, Paediatric and Neonatal ventilation thanks to an adjustable - minimum Tidal volume from 2 ml to 3000 ml
- PEEP up to 50 cmH₂O and Flow By leakages compensation, with NIV availability
- 9" TFT LED display, five graphs displayed simultaneously
- Three loops available P-V; P-F; F-V
- Compatible with premixed HELIOX
- External remote control panel to facilitate the therapy inside mono-place chambers
- Small dimensions and light weight (5,5 Kg)
- Special hermetic battery

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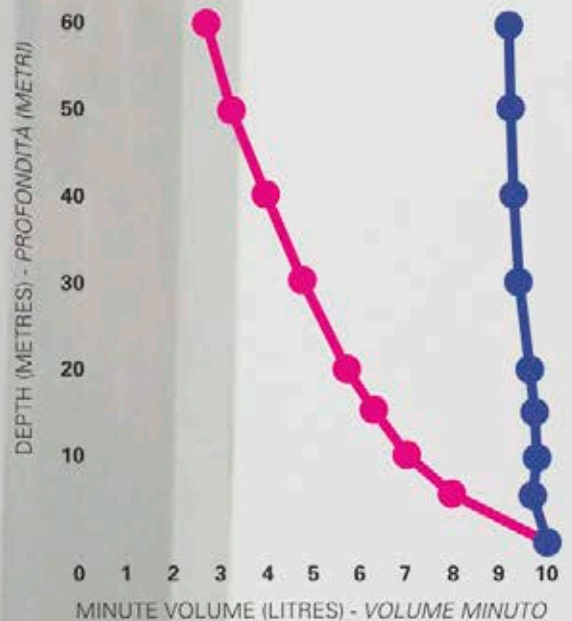
Code A56.NNT NEPTUNE HBOT

Code 05048
Castor LT trolley for
ventilator

Code 06750/SB
Stainless steel
patient circuit
supporting arm
with clamp

Code 05048/IPER
Castor LT trolley with
additional battery
and monitor support

Code G00198000
Ventilator plate
support for IV pole

Code G00198010
Monitor plate
support for IV pole

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Distributed by:

Data subject to change without notice - Revised on OCTOBER 2014 - DU0010530